

General Cleaning Instructions

Ocura are proud to supply furniture that is designed to withstand demanding environments. With the correct care and attention, your furniture will provide years of useful life. For optimum care and cleaning please read the following instructions and recommendations.

All General Surfaces

- Clean all surfaces with a soft, damp cloth and polish completely dry. Abrasive cloth must not be used.
- Any stain, spills or soiling should be cleaned up immediately to prevent the possibility of permanent staining.
- For disinfection, use diluted chlorine products at a strength of 1,000ppm chlorine and 10,000ppm for blood/blood stained body fluid spillages.
- All surfaces must be kept dry at all times. Exposure to wet or humid conditions could result in rust marks, paint lifting, discolouration or corrosion.
- Do not use concentrated bleach, synthetic detergents, solvents, wax polishes or aerosol sprays on any surface.

Vinyl Upholstery

Our vinyl ranges offer excellent stain resistance and cleanability, and have been tested for compatibility with the Department of Health and Social Care's recommended cleaning regimes. Our vinyl's are compatible with most NHS approved cleaning solutions and wipes such as Chlor-Clean, Haz-tab, Actichlor Plus, Milton, Clinell and Tristel. The vinyl should be cleaned daily in order to maintain its appearance and prevent build-up of dirt and contaminants.

- Clean the entire surface daily using a mixture of 1:9 liquid soap (neutral pH) and water with a clean, soft cloth. Abrasive cloth must not be used.
- Completely remove excess cleaner with a white, clean, damp cloth. All surfaces must be thoroughly dried.
- Any stain, spills or soiling should be cleaned up immediately to prevent the possibility of permanent staining.
- Do not use soap or cleaning solutions that contain alcohol, ketones, xylene, acetates or solvents (mineral/white spirits).
- Do not use solvents, abrasives, synthetic detergents, wax polishes or aerosol sprays.
- For disinfection, use diluted chlorine products at a strength of 1,000ppm chlorine and 10,000ppm for blood/blood stained body fluid spillages. Rinse and dry all surfaces immediately.

Electrical Components

- Electrical components must not be exposed to any form of liquid or humid conditions.
- Plastic casings containing actuators or handsets should be cleaned using a soft damp non-abrasive cloth and polished dry immediately.
- Do not use solvents, abrasives, synthetic detergents, wax polishes or aerosol sprays

Please note: Using cleaning products contrary to the manufacturer's instructions may compromise the integrity of the furniture and may invalidate your warranty.