

## CA4860/CA4861/CA4862 curaCARE Easy Clean Overbed Table/Chair/Compact



Every design brief has a purpose for us, when we started the process our goal was to create the best value Over Chair/Chair Table. The range is an essential part of patient care, and so our driver was to create a robust and durable product that was designed especially for demanding healthcare environments. The design ensured a product that is 'one wipe' clean' to aid infection control, the 50mm castors allow the table to be easily moved. The table is height-adjustable which allows the patient to choose the height to suit their needs.





## INSTRUCTIONS FOR USE



① To raise the table, lift the lever at the top of column, to lower the table apply downward pressure to the tabletop while the lever is lifted.

② The base has 4 x 50mm castors with 2 braked.



## HEALTH AND SAFETY



Max load evenly distributed 25KG.



Ensure there are no heavy objects on the table before activating the height adjustment.



The overbed table should in no way be used as support when standing or stepping.



There should be no heavy objects placed on the tabletop.